

# GO IMPLANT-FREE

SeQuent® Please

**Clinically proven** alternative to  
DES in de novo lesions

## ADVANTAGES OF DCB-ONLY


### No unnecessary stent implantation

- No inflammation due to a foreign body implant
- No risk of stent thrombosis
- No stent-related limitations for further treatment
- No stent edge effect

### Efficacy of DCB

- Enable positive remodeling<sup>1</sup>
- Keep natural vessel vasomotion
- Only 1-month DAPT<sup>2</sup>

# DCB-only Treatment Methodology<sup>1</sup>

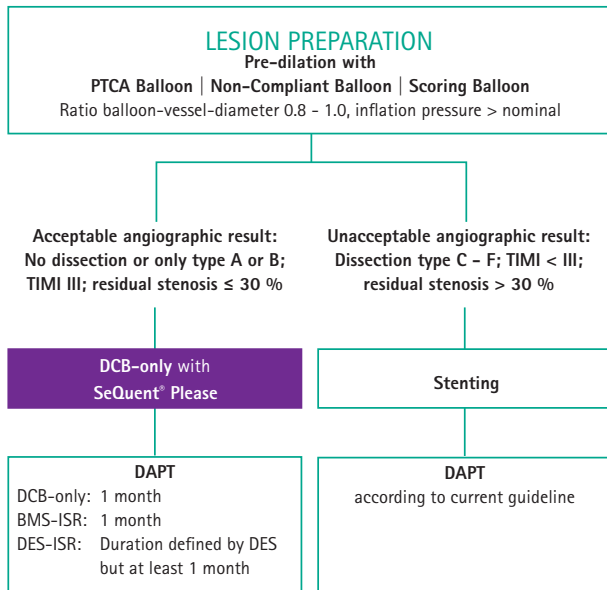

## Important handling aspects SeQuent® Please:

- Predilate the target lesion!
- Intended for local drug delivery
- DCB distal and proximal at least 2 - 3 mm longer than pre-dilated area
- Ratio balloon-vessel-diameter 0.8 - 1.0
- Inflation pressure 8 - 10 atm, time 30 seconds
- Do not bend, touch or wipe
- Do not expose any fluids to the catheter at balloon portion
- Position at the lesion site as fast as possible
- For single use only

1 Kleber FX et al. Local paclitaxel induces late lumen enlargement in coronary arteries after balloon angioplasty. Clin Res Cardiol. 2015 Mar 104(3):217-25