

Aesculap Surgical Technologies

PROPER MAINTENANCE OF INSTRUMENTS

Sterilit® - Maintenance Oils

- Base: paraffin/white oil.
- Reduces friction and protects against fretting corrosion.
- Guarantees good spreading and lubricating characteristics.
- Physiological harmlessness checked according to ISO 10993.¹
- Environmentally friendly (CFC-free).
- Suitable for steam sterilisation methods in conformance with ISO 17665-12 and hot air sterilisation up to 180°C.

Steam Penetration

The special formula ensures that the steam penetrates the oil film during the sterilisation process, even in areas difficult to access.

Properties

In accordance with the requirements of the Instrument Preparation Working Group (A.K.I.)³, Sterilit® possesses the following properties:

- Steam penetrable oil film.
- Oil film does not interfere with the effectiveness of the sterilisation.
- Lubricates and protects against corrosion.
- Physiological harmlessness checked according to ISO 10993.
- Silicone-free, no staining or crust formation on instrument surfaces.

Safe Handling

- Use before sterilisation only.
- Store in a cool, dry, dark place.
- Do not use after the expiry date.
- Medical devices must be prepared according to the instructions for use before Sterilit® is applied.

Remarks

Sterilit® carries the CE label and fulfils the fundamental requirements in compliance with MDD93/42/EEC.⁴

Scissors



Oiling a screw joint

Needle Holders




Oiling a box joint

Clamps & Forceps



Oiling a box joint

Rongeurs

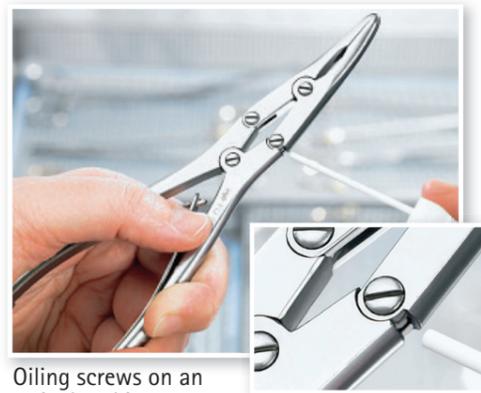

Oiling screws on an articulated instrument

Micro Instruments



Oiling a micro needle holder

Endoscopic Needle Holders



Oiling a ratchet

Bone Punches & Rongeurs



Oiling a bone rongeur

Bone Punches & Rongeurs



Oiling a bone rongeur



Aesculap Sterilit® Oil Spray and Instrument Oil

1. ISO10993-12: 2012 Biological evaluation of Medical Devices, pt 12. www.iso.org | 2. ISO17665-1: 2006 Sterilization of health care products. www.iso.org | 3. Reprocessing of Instruments to retain value. 10th edition 2012. www.a-k-i.org | 4. MDD93-42-EEC Medical Device Directive 1993

For further information please refer to full Proper Maintenance of Instructions (C71002)