

Ecoflac® Mix Drug Reconstitution



1. Removal of foil seal.



2. Attach Ecoflac® Mix to Ecoflac® plus.



3. Attach vial to Ecoflac® Mix.



4. Invert Ecoflac® plus and squeeze to reconstitute drug.
Agitate solution according to standard practice.



5. Revert and squeeze medication back into
Ecoflac® plus. Repeat as necessary.



6. Remove Ecoflac® Mix and vial to attach giving set.

For further information please contact your local B. Braun Representative or Customer Service on 1800 251 705