



Open pouch and unwrap set



Locate middle fold of the sleeve by looking for the "Hand here" sticker



Insert hand into the cover



Apply a small amount of sterile gel on either sleeve (a) or on the probe (b)



Pick up the Ultrasound transducer (a*) or have it be handed to you (b)
*Note: "probe here" visible for STOP (Check) variants only



Pull the cover over the transducer and cable ("pull here" sticker)



Fixation with rubber bands



Perform time out and remove peel off sticker (STOP (Check) variants only)



For clear ultrasound images make sure that there are no air bubbles under the probe cover



B. Braun Australia Pty Ltd | Norwest Business Park NSW, 2153
Tel. 1800 251 705 | info.au@bbraun.com | bbraun.com.au

B. Braun New Zealand Pty Ltd | Level 8, 139 Quay Street, Auckland, 1010
Tel. 0800 227 286 | info.nz@bbraun.com | bbraun.co.nz

BAUS HC E1308 11/25 | © 2025 B. Braun Australia Pty Ltd. ABN 56 002 945 155. All rights reserved.
EZCOVER™ is a trademark of B. Braun Medical S.A.S. (Germany). This brochure is intended for use by healthcare professionals only. It is provided for informational purposes and does not constitute medical advice. Always refer to the product's Instructions for Use and consult a qualified healthcare provider for clinical decisions.