



ASKINA® DERM

POLYURETHANE FILM DRESSING

B. Braun Wound Care. Right. From the start.

Askina® Derm

Polyurethane film dressing

Innovative application system

Maintains a moist environment

High MVTR film: 600 - 1000g/m² / 24hrs

Low tack adhesive: non aggressive removal

Impermeable to liquids and bacteria

Transparent: allows visual inspection of the wound

Prevents skin breakdown due to friction or continuous exposure to moisture

Can be used for the fixation of IV catheters

Can be used as a secondary dressing



1. Remove the printed release paper from the adhesive side of the dressing.



2. Place the dressing over the wound site with the tabs on the adhesive side facing toward the wound. Gently apply pressure to the dressing as it attaches to the wound.




3. Slowly peel away the tabs and gently apply the remainder of the dressing to the surrounding dry skin.



4. Remove the plastic carrier by peeling the tabs in the direction of the arrows.

INDICATIONS

- Minor burns
- Donor sites
- Pressure areas
- Post-operative wounds
- Abrasions and lacerations
- Superficial acute wounds

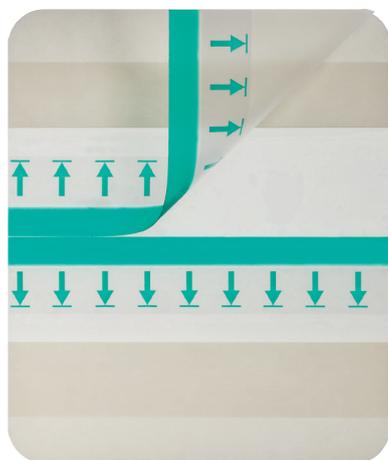

Askina® Derm is a sterile transparent polyurethane film dressing. It is semipermeable and suitable for the protection of the newly formed skin. Prevents scab formation. It is also used as protection against skin breakdown due to friction or continuous exposure to moisture.

B. Braun Ordering Information

Askina® Derm

Size	Pcs/Pack	Article number
6 x 7 cm	10	F72032
6 x 7 cm	100	F72033
10 x 12 cm	5	F72034
10 x 12 cm	10	F72035
10 x 12 cm	50	F72036